Pressotherapy
Pressothérapie
Presoterapia

mainat

mainat

Pressotherapy

Fabric:

- Soft to the touch.
- Several colours available, depending on type of garment.

Dressmaking:

- Each piece is personally and individually made.
- The few seams are exterior and completely flat to avoid rubbing in sensitive areas.
- Natural anatomic shape formed with darts.
- Openings can be made to aid placement.

Pressure:

- Levels of pressure controlled at 15 to 50 mm Hg.
- Dynamic pressure 100% from recovery of elasticity.
- Levels of pressure of the garment adapted to patient's needs.
- Varying levels of pressure possible between different parts of the same garment.

Service:

- Delivery: 10-15 working days
- Technical support
- Product training

Washing and instructions of use:

- Avoid oil and grease stains, which result in garment damage.
- Wash only with neutral soap and do not expose wet garments to direct sunlight or dry them close to a heater.

Guarantee:

- Garments for adults: 6 month guarantee following washing and instructions of use.
- Garments for children: 3 month guarantee following washing and instructions of use.
- Any garments altered outside Mainat will be exempt from guarantee

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The colors of the garments may vary from the pictures and the samples on the catalogue.

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Available colors
Couleurs disponibles
Colores disponibles

JM.003 / JM.004

Child's sleeveless vest (0-14 years) / Adult's sleeveless vest

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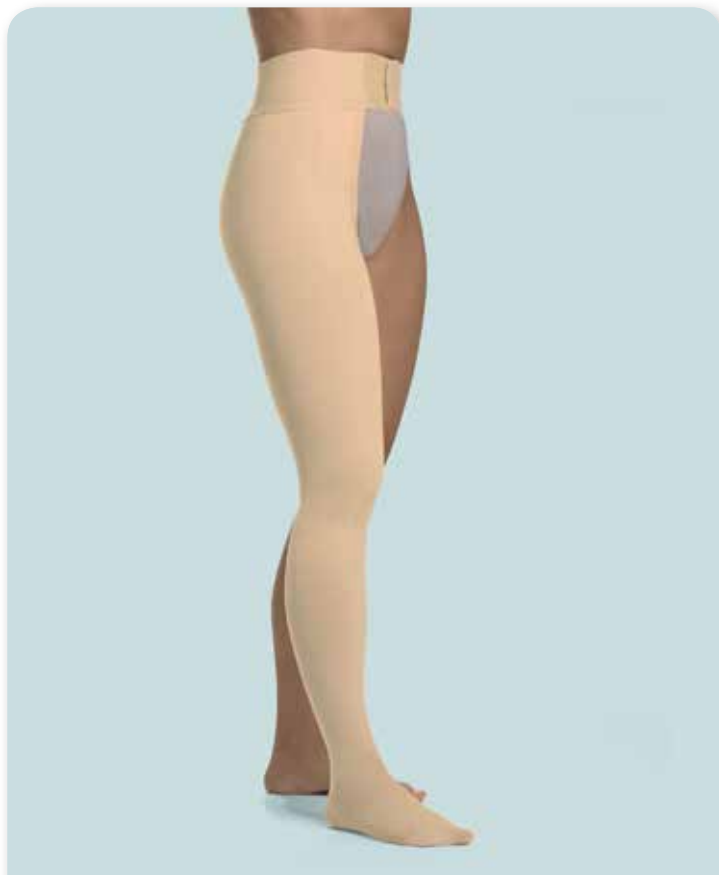
Available colors
Couleurs disponibles
Colores disponibles

JM.005.2

Axilla-length sleeve with gauntlet

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Tubular brazo guantelete

Available colors
Couleurs disponibles
Colores disponibles

JM.007 / JM.008

Child's sleeve with shoulder strap (0-14 years) / adult's sleeve with shoulder strap

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Available colors
Couleurs disponibles
Colores disponibles

JM.024

Adjustable head band
Couvre front
Banda frente regulable

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Available colors
Couleurs disponibles
Colores disponibles

JM.026

Full front face mask (nose piece optional).

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Available colors
Couleurs disponibles
Colores disponibles

JM.26.B

Face mask, chinless (nose piece optional)

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Available colors
Couleurs disponibles
Colores disponibles

How to place an order for custom-made garments

We provide you with six measurement forms, depending on the type of garment your patient requires

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Patient Age

Orthopaedics / Distributor VAT number

Prescribing Doctor / Centre Member number

Address

Post code

 Town Province

Telephone

 Fax

 e-mail

☐ **JM.001** Child's vest (0-14 years)

☐ **JM.002** Adult's vest

☐ **JM.002.S** Women's shaped vest

☐ **JM.003** Child's sleeveless vest (0-14 years)

☐ **JM.004** Sleeveless vest

☐ **JM.005** Axilla-length sleeve

☐ **JM.005.2** Axilla-length sleeve with gauntlet

☐ **JM.007** Child's sleeve with shoulder strap (0-14 years)

☐ **JM.008** Sleeve with shoulder strap

Quantity

Have you ever ordered the same item before ?

☐ YES ☐ NO

Opening

☐ Front ☐ Back

Remarks

VERY IMPORTANT:

Mark injuries on the drawing. Take measurements with a standard measuring tape and make sure you do not omit any of the required measurements - please ensure that they are accurate.

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Enter patient's measurements

Measurements should be taken first thing in the morning.
Measurements should be taken with a metric tape measure.
Make sure you fill them all.
We recommend putting the garment on the patient at around the same time as the measurements were taken.

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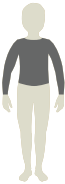
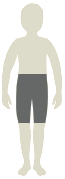
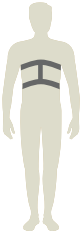
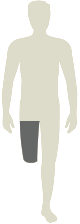
Mainat is a family business with more than 30 years experience in the sector.

We make medical textile garments for two specialities: made-to-measure pressotherapy for treatment of hypertrophic scarring after burns, vascular disorders and lymphoedemas, and standard size pressotherapy for postoperative treatment after plastic or aesthetic surgery.

Made-to-measure production involves detailed studies of each garment and direct contact with doctors, therapists or orthopaedic technicians to guarantee the best treatment for the patient and a high level of comfort. The result: unique personalized products.

Mainat commercializes its products in over 40 countries through distributors.

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J. Mainat, S.A.